Caso clínico Clinical Case

11

Fase Quirúrgica y Protésica
/ Surgical and prosthetic Phase

Colaboración / Collaboration:
Dr. Victor Cubillo Blasco

Clínica Artedental - Tenerife

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- Gender: Male
- Age: 73 years old
- Treatment: ArtOn4® Immediate maxilla loading rehabilitation on four implants and bone graft with Beta-tricalcium phosphate.
- Type and implant size: ZinicMT

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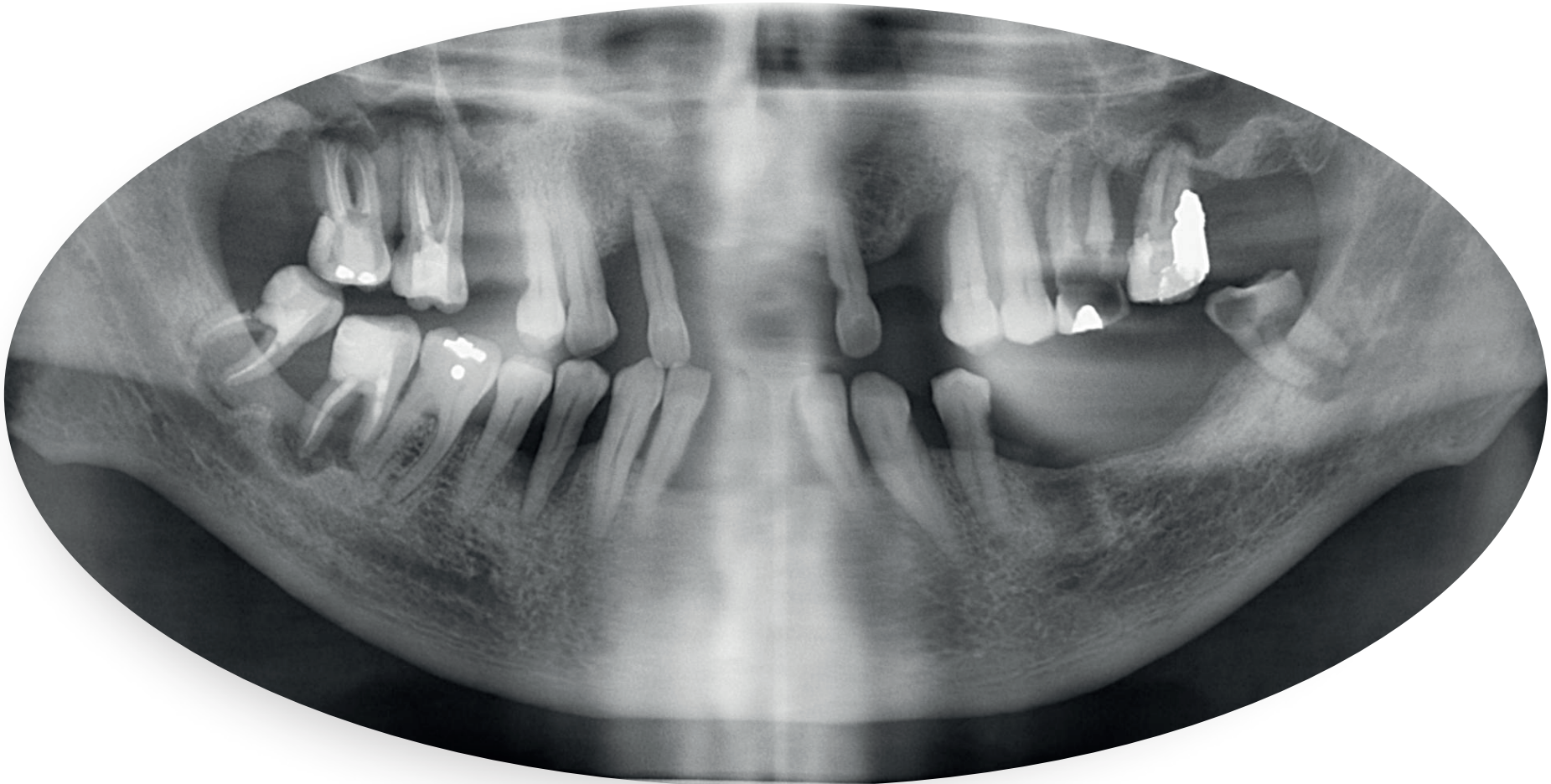
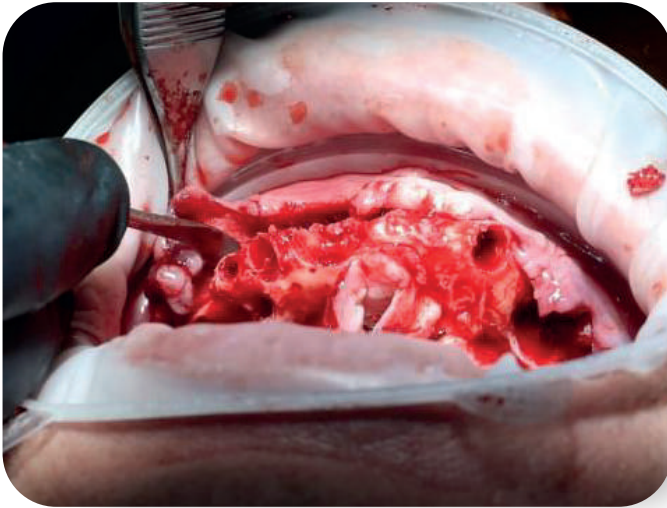
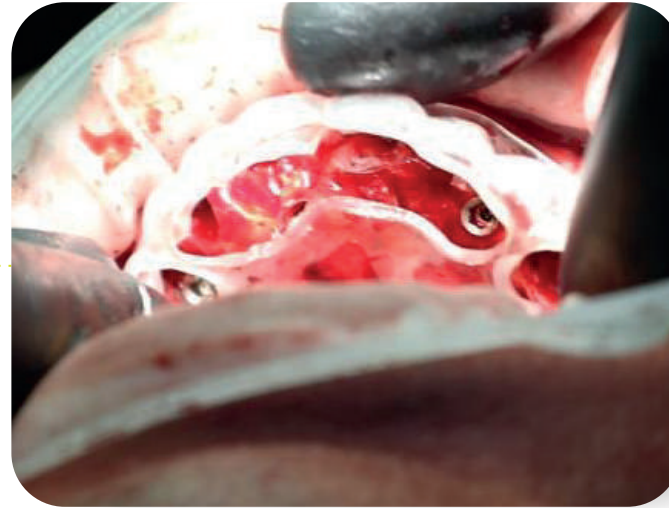
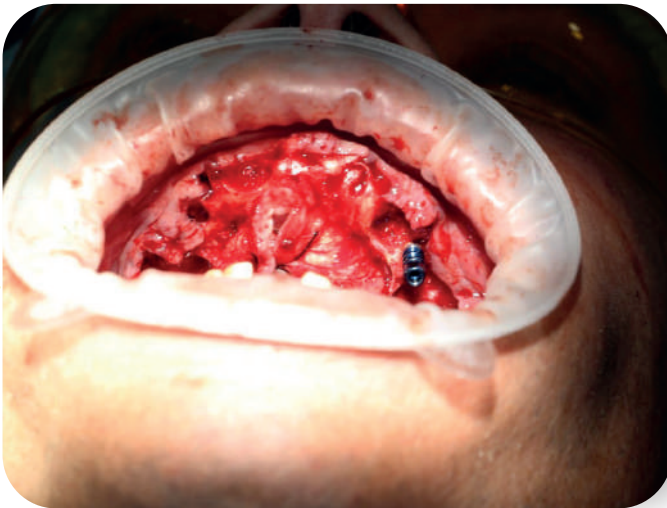
Connection

Compatible hexagonal internal connection Zimmer[®](1) (See)
Depth of the internal hexagon (2,0mm): longitudinal forces distribution
Conical bevel: Reduces infiltration
Friction fit: reduces micromovement
Platform switch: better representation of biological width, long term aesthetic stability.

Cortical area

Mechanized ring (0,4mm): allows to raise the prosthetic gap according to the crestal bone in medium-thick biotypes; avoiding to expose the treated surface in irregular crest.
Micro-thread design: preserves the marginal bone, reduce bone loss
Micro-thread extension: improves load distributions
B» Macro design: makes an appropriate cortical compression.
Beveled shoulder and mini-platform: allow horizontal biological sealing.

Body

Double lead thread of reduce angle: provide stability during insertion and increase bone implant contact (BIC)
Active apex (self-tapping): facilitates insertion in under drilling
Apical vents: collect bone detritus during the insertion.
Optimized morphology: excellent primary stability.
Atraumatic apex: protect anatomical structures.

Tapered Desing

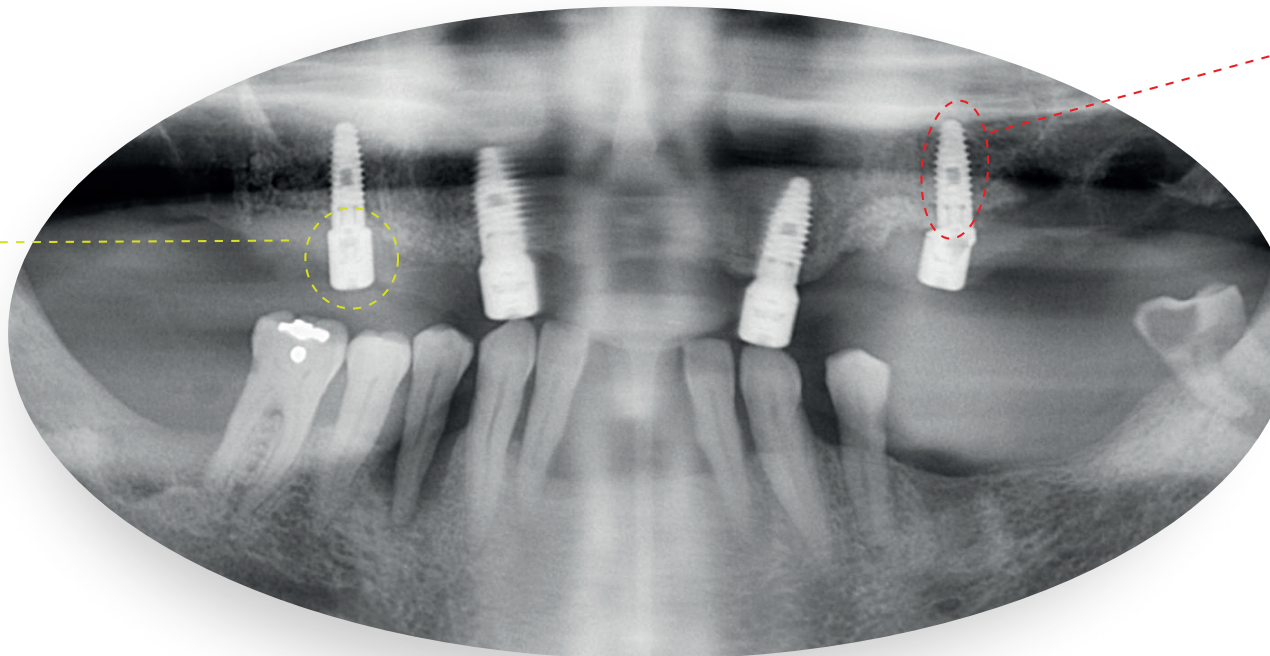
Facilitates insertion during soft bone placement.
Suitable for immediate loading.
For apical collapse situations.

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
/ Placement of healing screws during the production of Hybrid (Titanium + Composite) prosthesis for 24-hour loading.

RADIOGRAFÍA POSTQUIRÚRGICA / POST SURGICAL X-RAY

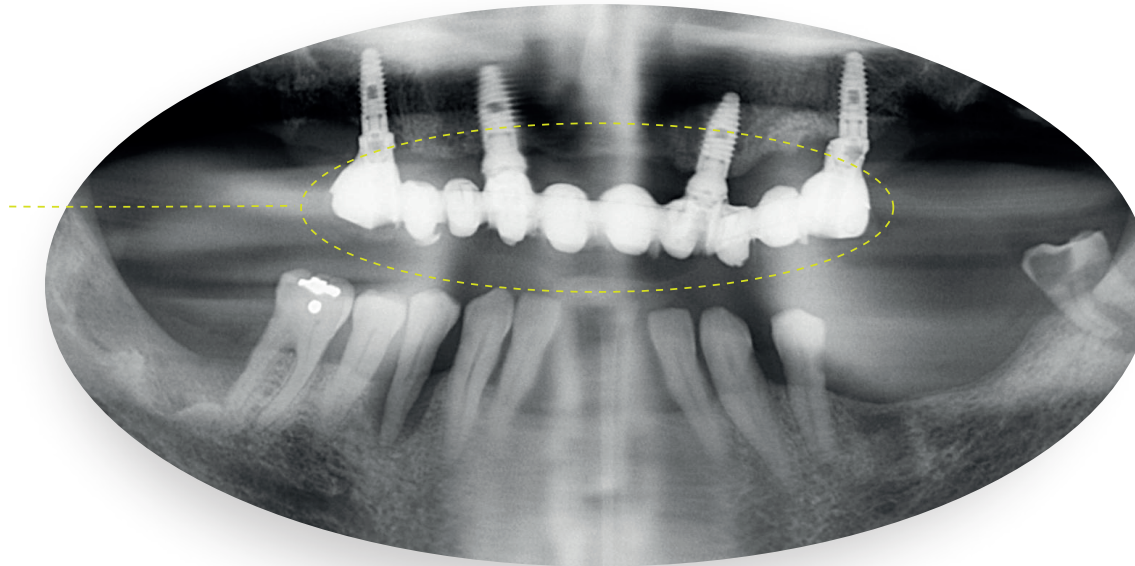Verificación de ajuste
de estructura /
Structure Adjustment
Verification


PRÓTESIS HÍBRIDA TITANIO Y COMPOSITE / TITANIUM HYBRID AND COMPOSITE PROSTHESIS

Caracterización de color y forma natural
de acuerdo a la estructura facial
/ Characterization of color and
natural form according to facial structure

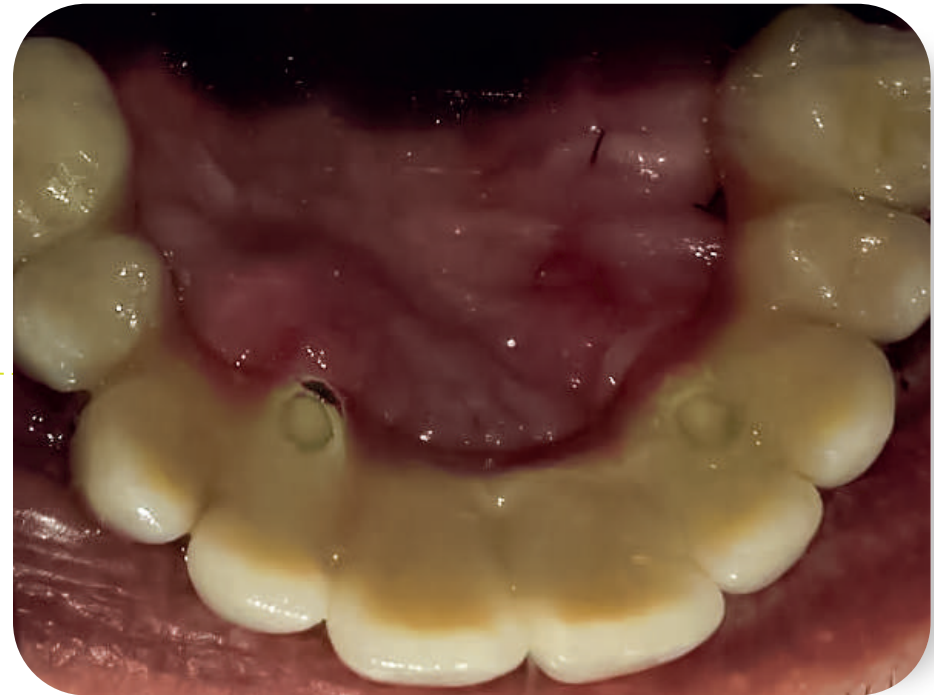PRÓTESIS HÍBRIDA TITANIO Y COMPOSITE / TITANIUM HYBRID AND COMPOSITE PROSTHESIS

ANTES Y DESPUÉS / BEFORE AND AFTER

Ziacom[®]

Together for **health**