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EN | INSTRUCTIONS FOR USE

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FR | INSTRUCTIONS D'USAGE

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Ziacom®, Basic®, Di2gital Arch®, DSQ®, Galaxy®, Kiran®, Kirator®, OsseosBCP®, OsseosTCP®, PlexGuide®, Titansure®, Titansure® Active, Tx30®, XDrive®, Zellplex®, ZiaCam®, Ziacom®3D, Ziacor®, Zinic®, Zinic®MT, Zinic®Shorty, ZiTium®, ZM1®, ZM4®, ZM4®MT, ZM8®, ZM8®N, ZM8®S, ZM-Equator®, ZMK®, ZMR®, ZPlus®, ZV2® y todos sus logotipos son marcas registradas de Ziacom Medical SL.

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### Ziacom® ZS1® Fixation pins

#### INSTRUCTIONS FOR USE

Instructions for use for ZS1® fixation pins.

#### IMPORTANT INFORMATION

Please read these instructions carefully before using the product.

This document contains essential information on the use of Ziacom® original fixation pins, hereinafter "Ziacom® fixation pins" or simply "Ziacom® products". This documentation has been prepared as a quick guide to be consulted by the clinician responsible guided bone regeneration, hereinafter the "user", and does not provide sufficient guidelines or technical specifications for correct use of ZIACOM® products. This is not an alternative nor substitute for specialised training or professional clinical experience.

Ziacom® products must be used according to a suitable treatment plan and in strict compliance with the working protocols established by Ziacom Medical S.L. Please read the specific working protocols for this product, as well as the instructions for use and cleaning, disinfection and sterilisation carefully before using this Ziacom® product. You can find this information on our website at [www.ziacom.com](http://www.ziacom.com) or request it from your Ziacom Medical S.L. authorised distributor.

### 1. Description of the System

This Ziacom® product consists of fixation pins and surgical instruments.

Ziacom Medical S.L. has created two fixation pins that serve to secure and immobilise resorbable and non-resorbable membranes in bone regeneration treatments in accordance with the scientific literature and current clinical standards. Ziacom Medical S.L. also develops, manufactures and markets latest-generation implant surgery products and a wide range of restoration solutions to meet each patients particular needs.

### 2. Description of the Products

#### Fixation pin

Ziacom® fixation pins are manufactured in biocompatible grade 5 titanium alloy, in accordance with standards ASTM F136-13 and UNE-EN-ISO 5832-3:2017, without surface treatment to avoid excessive osseointegration that hinders explantation at the end of treatment. ZIACOM® fixation pins are manufactured with the same head in different diameters and lengths.

### 3. Indications for Use

Before carrying out treatment with Ziacom® products, it is vital to assess the patient beforehand with clinical diagnosis, imaging and study models, as well as determine possible contraindications for the treatment. Correctly performed bone regeneration treatment aids the formation of viable bone tissue for placement of a dental implant or to treat certain types of bone defect.

#### Fixation pin

Ziacom® Fixation pins are indicated for securing and stabilising the bone graft and the (resorbable or non-resorbable) membrane in guided bone regeneration treatments, which encourages the formation of bone tissue.

Insertion and removal of the fixation pin must be carried out by a qualified professional using the recommended kit and following the guidelines in the catalogues and working protocols.

### **Important**

Check the clinical indications of the fixation pins thoroughly in the corresponding catalogues and working protocols before planning treatment. Do not use the Ziacom® fixation pins in locations or for purposes other than as described in the indications in the catalogues or working protocols.

### **4. Contraindications**

The patient must undergo a preoperative medical examination, as well as assessment of the surgical conditions of the area and of the procedure to determine whether there are any absolute and/or relative contraindications or risk factors for the use of fixation pins. The user is responsible for evaluating the potential risks and benefits of the use of the fixation pins in bone regeneration procedures. Ziacom® products should not be used in patients who lack the minimal medical conditions necessary to carry out regeneration treatment. Neither should dental implants and prosthetic rehabilitations using dental implants be placed in patients who do not meet the minimum medical conditions.

#### **General Contraindications:**

Patients with systemic diseases (blood clotting disorders, endocrine disorders, rheumatic diseases, liver disease, neoplasms or other serious diseases), psychological or mental disorders, poor oral hygiene, untreated severe periodontal disease, toxic habits (smoking, drug addiction, alcoholism), active infection or pathological processes at the site of placement, allergies to the fixation pin materials, anatomical structures at a distance of less than 2 mm from the site of insertion (teeth, inferior alveolar canal, nasopalatine duct, maxillary sinus, etc.)

The user should review the compatibility of certain medications with bone regeneration treatment.

Ziacom Medical S.L. will update these contraindications as necessary, based on information obtained from clinical studies and/or post-marketing information.

### **5. Storage**

Ziacom® products must be stored in their original packaging in a dry, clean place away from sunlight and adverse conditions. They must be stored at a temperature below +55°C and above -10°C.

### **6. Packaging and Supply Format**

Ziacom® fixation pins are supplied in a cardboard box with a product identification label, which functions as a security seal to protect against tampering.

The pack contains:

- The QR code to access these explanatory instructions for use of the product.
- Plastic heat-sealed blister and removable labels on the back, indicating the manufacturer, full product reference, batch number, fixation pin dimensions and relevant symbols (explained in these instructions for use). The fixation pin is enclosed inside.

The removable labels are intended for traceability of the fixation pin, by sticking one label on the patient's history and the remaining ones on the clinical record.

Ziacom Medical S.L. guarantees that all of its products follow a strict manufacturing, monitoring and cleaning process before being packaged. Before using Ziacom® products, you must check that the packaging is intact and ensure that it is not damaged. If there is any damage or defect, the product must not be used and Ziacom Medical S.L. or your authorised official distributor must be notified.

### **7. Cleaning and Sterilisation**

Ziacom® fixation pins are supplied NON sterile.

Fixation pins that are to come into contact with the mouth must be cleaned, disinfected and sterilised before use, following the Ziacom Medical S.L. cleaning, disinfection and sterilisation instructions. You can find this on our website at [www.ziacom.com](http://www.ziacom.com) or in Ziacom Medical S.L. catalogues.

### **8. Precautions**

#### **Insertion and Instruments**

The working protocols describe in detail the precautions and important

factors to be taken into consideration during placement of the fixation pins. Prior to placement, it is essential to have consulted the working protocol for the product, along with all other indications regarding the correct use as indicated in the Ziacom® product catalogues.

Special care must be taken to take the necessary steps to prevent the fixation pin or instruments from being swallowed or aspirated by the patient.

Take particular care not to use excessive force during placement of the fixation pins, as well as when disconnecting the impactor from the fixation pin once it is placed in the desired position. If during placement a variation is detected in the bone consistency, stop insertion immediately and make sure that you are not inserting into inadequate bone or dental structure, for example into tooth roots, as well as if you detect any deformity in the body of the fixation pin that may lead to any type of fracture.

### **Please note**

The maximum insertion depth for the fixation pins is reached when the head is in contact with the membrane or graft, making sure that this remains immobile. Avoid applying excessive force with the mallet on the impactor, as well as ensuring that the force applied is perpendicular to the pin.

### **9. Warnings**

Planning regenerative treatment, selection of the biomaterials to use and placement of Ziacom® fixation pins requires specific training in dentistry. The product specifications themselves are no guarantee of proper use. We recommend that users take specialised theoretical and practical training courses in order to understand the proper working techniques and protocols for each product, including the biomechanical, radiographic and biological requirements associated with bone regeneration treatment.

It is the user's responsibility to know and find out about the current status of the technique for each product and its possible applications.

It is necessary to be familiar with and understand the corresponding working procedures before using Ziacom® products. The patient must meet sufficient medical, anatomical and psychological conditions to undergo treatment with dental implants. The user must ensure proper treatment planning is carried out, to guarantee the safety of the treatment with a minimal margin of error, respecting the patient's vital mouth structures and general health.

The procedure for using the insertion kit for placing fixation pins is specified in the working protocols corresponding to each type of fixation pin. Placement of the fixation pin and the bone regeneration procedures must be tailored to the conditions of the individual patient. Use or insertion of fixation pins other than as specified by Ziacom Medical, S.L. in the working procedures is not recommended.

Ziacom® fixation pins are single-use products. Reuse of single-use products carries the risk of possible deterioration of their characteristics. The geometry of the product may have suffered damage from previous use or due to inadequate use. This entails the risk of failure of the bone regeneration and/or damage to the patient's health.

### **10. Adverse Events or Side Effects**

Regenerative treatments may lead to various adverse or undesirable effects that are documented in the specialised scientific literature published in specialised journals or books on dentistry.

However, the most relevant are:

Laceration of the membrane, injury to adjacent anatomical structures (nasal passages, maxillary sinus, vascular/nerve structures), infections, entrapment of the pin, etc.

Ziacom Medical S.L. will update these side effects as necessary, based on information obtained from clinical studies and/or post-marketing information.

### **11. Liability, safety and warranty information**

The instructions for use and handling of Ziacom® products are based on internationally published literature, current clinical standards and our own clinical experience so should be understood as general guiding information. The handling and use of Ziacom® products is the sole responsibility of the user as it is outside the control of Ziacom Medical S.L. Ziacom Medical S.L., their subsidiaries and/or their official authorised distributors do not accept any liability, whether explicit or implicit, total or partial, for possible damage or injury caused by improper handling of the product or any other situation not considered in their protocols and instructions for the correct use of their products.

The user must ensure that the Ziacom® product is appropriate for the intended procedure and end purpose. Neither these instructions for use nor the working or handling protocols for the products release the user from this obligation. Ziacom® products must be used, handled and applied clinically by professionals with the appropriate training and qualifications required according to current legislation in each country. Total or partial use, handling or application of any phase of Ziacom® products by unqualified personnel, or by personnel that does not have the necessary training, will automatically void any type of warranty and may cause serious damage to the patient's health.

Ziacom® products have their own design features and working protocols, so the insertion of the fixation pins should be carried out with the adaptors and insertion kit recommended by Ziacom Medical S.L. in the catalogue and the working protocols. Insertion of Ziacom® fixation pins with adaptors or instruments from other manufacturers may lead to insertion failure, deterioration of the fixation pin, adverse impact on the area of insertion in causing damage to the tissues and bone structures. For this reason, insertion should be carried out with the kit and adaptors recommended by Ziacom Medical, S.L.

The design of the fixation pins enables them to be used together with other products, such as various types of membranes and biomaterials.

The clinician in charge of the treatment is solely responsible for ensuring that original Ziacom® products are used and that they are used according to the corresponding instructions for use and working protocols throughout the entire guided bone regeneration treatment. The use of any other non-original Ziacom® instruments, whether alone or in combination with any original Ziacom® products, will immediately void the warranty of the original Ziacom® products. See the Ziacom Medical S.L. Warranty Programme (available on the website or by contacting Ziacom Medical SL, their subsidiaries or authorised distributors).

## Warning

Not all Ziacom® products are for sale in every country. Check availability in your country.

Ziacom®, Basic®, Di2gital Arch®, DSQ®, Galaxy®, Kiran®, Kirator®, OsseosBCP®, OsseosTCP®, PlexGuide®, Titansure®, Titansure® Active, Tx30®, XDrive®, Zellplex®, ZiaCam®, Ziacom®3D, Ziacor®, Zinic®, Zinic®MT, Zinic®Shorty, ZiTium®, ZM1®, ZM4®, ZM4®MT, ZM8®, ZM8®N, ZM8®S, ZM-Equator®, ZMK®, ZMR®, ZPlus®, ZV2® and all their logos are registered trademarks of Ziacom Medical S.L.

The Ziacom® brand is a registered trademark, as are some of the products that may or may not be mentioned in this document. Ziacom Medical, SL reserves the right to modify, change, eliminate or update any of the products, prices or technical specifications referenced in this document or in any of its catalogues without prior notification. All rights reserved. The reproduction of this document, whole or in part and in any medium or format, without the corresponding written authorisation from Ziacom Medical SL is prohibited.

## 12. Magnetic Resonance

The safety and compatibility of Ziacom® products in a magnetic resonance environment has not been evaluated, nor have they undergone testing to verify whether they heat up or migrate in such environments.

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Ziacom®, Basic®, Di2gital Arch®, DSQ®, Galaxy®, Kiran®, Kirator®, OsseosBCP®, OsseosTCP®, PlexGuide®, Titansure®, Titansure® Active, Tx30®, XDrive®, Zellplex®, ZiaCam®, Ziacom®3D, Ziacor®, Zinic®, Zinic®MT, Zinic®Shorty, ZiTium®, ZM1®, ZM4®, ZM4®MT, ZM8®, ZM8®N, ZM8®S, ZM-Equator®, ZMK®, ZMR®, ZPlus®, ZV2® e todos os seus logótipos são marcas registadas da Ziacom Medical SL.

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#### Rx Only

**Caution:** federal law prohibits dispensing without prescription.

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Symbols of labelling codes and used in catalogues, standard ISO 15223-1

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