



Instructions for the download and installation of the ZIACOM 3shape libraries

Download software

From your computer, open the browser and enter the following URL: www.ziacom.com

In the DIGITAL FLOW menu, go to the section CAD/CAM libraries > select 3Shape.

Click on **3shape** 

A file called 3Shape.zip will be downloaded to the Downloads folder on your computer.

Decompress the file

Go to the Downloads folder on your computer and decompress the file called

3Shape.zip.


To decompress the file, you can use a standard decompression tool that can be downloaded online such as 7-zip.

When you decompress the file, a folder with the following content will appear:

- ZIACOM_D.IMPLANT(+0.05)_1.0.dme
- ZIACOM_D.IMPLANT(+0.10)_1.0.dme
- ZIACOM_D.IMPLANT(+0.15)_1.0.dme
- ZIACOM_D.IMPLANT(-0.05)_1.0.dme
- ZIACOM_D.IMPLANT_1.0.dme
- ZIACOM_TIBASE(+0.05)_2.1.dme
- ZIACOM_TIBASE(+0.10)_2.1.dme
- ZIACOM_TIBASE(+0.15)_2.1.dme
- ZIACOM_TIBASE(-0.05)_2.1.dme
- ZIACOM_TIBASE_2.1.dme

Import of libraries

1. Open the 3Shape Control Panel from the shortcut or from Dental Manager (nut icon in both cases).

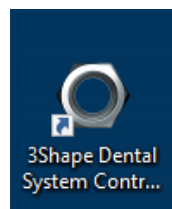

2. Then, click on the "Import/Export" option in the Tools menu.




3. Once inside the option, click on the button “Import materials”.




4. Now you must select the file you need from the decompressed folder downloaded previously.

	ZIACOM_D.IMPLANT(+0.05)_1.0.dme	28/11/2022 9:14	Archivo DME	4.246 KB
	ZIACOM_D.IMPLANT(+0.10)_1.0.dme	28/11/2022 9:15	Archivo DME	4.246 KB
	ZIACOM_D.IMPLANT(+0.15)_1.0.dme	28/11/2022 9:17	Archivo DME	4.247 KB
	ZIACOM_D.IMPLANT(-0.05)_1.0.dme	28/11/2022 9:18	Archivo DME	4.247 KB
	ZIACOM_D.IMPLANT_1.0.dme	28/11/2022 9:12	Archivo DME	4.249 KB
	ZIACOM_TIBASE(+0.05)_2.1.dme	28/11/2022 9:22	Archivo DME	20.801 KB
	ZIACOM_TIBASE(+0.10)_2.1.dme	28/11/2022 9:23	Archivo DME	20.697 KB
	ZIACOM_TIBASE(+0.15)_2.1.dme	28/11/2022 9:25	Archivo DME	20.699 KB
	ZIACOM_TIBASE(-0.05)_2.1.dme	28/11/2022 9:26	Archivo DME	20.698 KB
	ZIACOM_TIBASE_2.1.dme	28/11/2022 9:20	Archivo DME	20.707 KB

5. Once selected, the following window will appear, where you must click on the button “Import”.

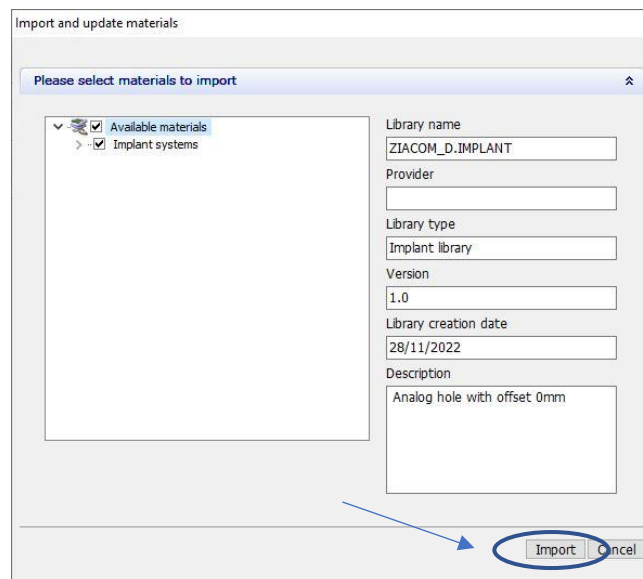

6. Once the process is complete, a window will appear in which you must select the button “OK” (Accept).

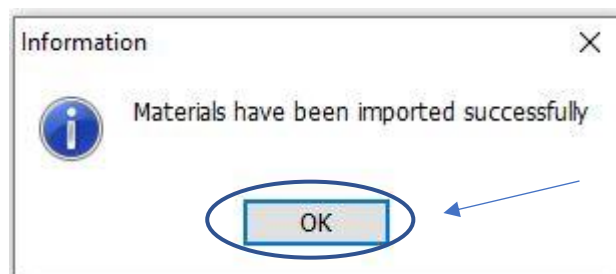

7. Upon completion, click on the button “Save” to the top left of the screen (disc icon).



8. Click “Yes” in the following window to confirm that you want to save the changes.

